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Intended purpose:

- Stress incontinence in women
- **Prolapse**
- Mixed incontinence with overactive bladder
- Recurrent incontinence after surgery
- Prevention in association with physical exercise (workout)

Material data:

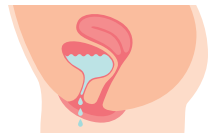
Tampon: PVA foam, string: cotton, applicator: polystyrene

Directions for use:

- 1 Wash hands thoroughly. Open the film along the perforated line, remove the tampon and unwind the removal string.
- 2 Soak Contam® briefly in warm water (approx. 1 minute), squeeze out remaining liquid. Contam® is soft and ready for use. Contam® is inserted in the same way as a menstrual tampon.

3 Use for bladder weakness

With your fingertips, insert Contam® deep into your vagina. Then pull it back slightly so that you can touch it with your fingertips.

Vagina, bladder and urethra area without tampon (urine leakage).

The tampon in the vagina exerts pressure on the urethra, thus preventing the uncontrolled discharge of urine. Put the tampon in a position that is comfortable for you.

The tampon has been inserted too deep into the vagina!

3 Use for prolapse problems

With your fingertips to insert Contam® as deeply as possible into your vagina. The tampon cannot slip away inside your body!

Vagina, bladder and urethra area without tampon.

The tampon has been inserted deep enough and therefore prevents prolapse.

The tampon has not been inserted deep enough into the vagina!

3 Using Contam® with an internal hole

3 Place the tampon with the internal hole onto the spike of the applicator and then insert the tampon.

4 If it does not feel comfortable, please move the tampon into a comfortable position with your fingertips. You cannot feel Contam® when it is positioned correctly. The end of the retrieval string must be outside the vagina.

5 To remove Contam®, you should adopt a comfortable position. Find the best direction for removing the tampon by gently pulling on the retrieval string.

6 Dispose of Contam® in household waste; **do not throw it into the toilet**. Wash your hands afterwards.

Using the toilet: There is no need to remove the Contam® vaginal tampon when urinating.

Med SSE does not accept any liability for damages arising from non-adherence to the instructions for use.

Safety notes:

- You should change Contam® after swimming/bathing.
- In the unlikely event of the retrieval string breaking despite our careful monitoring through continuous inspection, press while squatting (like for a bowel movement), try to get hold of the tampon with two fingers and remove it. Should that be unsuccessful, go and see a doctor immediately.
- Contam® is a disposable product and must not be reused. Hygiene requirements are no longer met if the product is reused!
- Contam® must not remain inside the vagina for more than 12 hours. Patients who cannot ensure compliance with this requirement must not use Contam®.
- If you suddenly experience fever, diarrhoea, vomiting, bleeding, dizziness or skin irritation while using Contam®, you must remove the tampon immediately and see a doctor.
- Do not use Contam® after the expiration date. Keep away from direct sunlight and heat; store in a dry place.

Contraindications (do not use Contam®):

- PVA allergy
- menstruation
- while sleeping or during infections
- seek medical advice before use in pregnancy

Side effects:

There are no known side effects

Every serious incident that occurs in relation to the medical device must be reported to the manufacturer, who will report this to the responsible authority.

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